

DIRECTIONS FOR CPAP AND BIPAP

1) You will need to purchase a 12-volt DC power cord, unique to your make and model of CPAP/BIPAP machine. DC Power cords are available from your CPAP/BIPAP medical supplier or from EXPION360 at www.expion360.com or call our office at 541-797-6714.

2) Plug one end of the 12 volt DC power cord into your CPAP/ BIPAP machine and the other end into the large power port marked 12V/15A on the battery bank.

3) The battery bank will automatically turn on and provide power to your machine.

QUICK TIPS

To determine the number of hours of run time you will get using your CPAP/ BIPAP machine connected to the EXP 38Pro or EXP 72Pro, you will need to refer to your manufacturer's specifications to determine the amount of amperage your machine uses at different pressure settings.

To get the most run time from the battery bank, we recommend turning off all auxiliary features such as humidity and heated hose. These features use a large amount of amperage and will draw most of the available power from the battery bank. Make sure your mask has no air leaks. This additional pressure will greatly increase the power draw from the battery and will shorten the amount of nights it will run your CPAP/ BIPAP machine.

The EXP PRO battery bank was designed to power most machines 2-3nights. However, power needs can vary greatly from patient to patient and from different manufacturers. We recommend you test your machine with the battery prior to any sleep use to fully understand the potential of your battery bank with you machine.

QUESTIONS: SUPPORT@EXPION360.COM